

It's your time to shine



PRO-SYS™

PROFESSIONAL

TOOTH WHITENING SYSTEM

Take Home Kits

Carbamide Peroxide
Hydrogen Peroxide

PRO-SYS

PROFESSIONAL

TOOTH WHITENING SYSTEM

TAKE-HOME KITS

Pro-Sys™ Professional take-home whitening kit delivers a high-performance system for a more radiant smile. Available in your choice of carbamide or hydrogen peroxide formulas, Pro-Sys™ supports a wide range of uses. To eliminate sensitivity, Pro-Sys™ contains potassium nitrate and its water-based gel keeps teeth hydrated.

HYDROGEN PEROXIDE

Patient Kit

(4) 3.5ml Syringes,
(1) Tray Case,
(2) EVA sheets
8% [4767-540]
14% [4767-568]



CARBAMIDE PEROXIDE

Patient Kit

(8) 1.2ml Syringes,
(1) Tray Case,
(2) Universal Trays
16% [4731-766]
22% [4731-775]
35% [4751-673]



HYDROGEN PEROXIDE

Refill Kit

(2) 3.5ml Syringes
8% [4767-577]
14% [4767-586]



CARBAMIDE PEROXIDE

Refill Kit

(4) 1.2ml Syringes
16% [4751-691]
22% [4751-708]
35% [4751-717]



HYDROGEN PEROXIDE

Bulk Kit

(20) 3.5ml Syringes
8% [4781-686]
14% [4781-695]



CARBAMIDE PEROXIDE

Bulk Kit

(40) 1.2ml Syringes
16% [4731-784]
22% [4731-793]
35% [4757-033]



WEAR TIMES

8% HYDROGEN PEROXIDE SYRINGE

Daytime use: Wear 30 minutes, once or twice a day.
Allow a minimum of one hour between wear time.
Do not exceed 30 minutes per application.

14% HYDROGEN PEROXIDE SYRINGE

Daytime use: Wear 5-10 minutes, once or twice a day.
Allow a minimum of one hour between wear time.
Do not exceed 10 minutes per application.

If sensitivity occurs, discontinue use for 24 hours.

16% CARBAMIDE PEROXIDE SYRINGE

Daytime use: Wear 4-6 hours, once per day.
Nighttime use: Wear 8-10 hours, once per day.

22% CARBAMIDE PEROXIDE SYRINGE

Daytime use: Wear 60 minutes, once or twice a day.
Allow a minimum of one hour between wear time.
Do not exceed 60 minutes per application.

35% CARBAMIDE PEROXIDE SYRINGE

Daytime use: Wear 20 minutes, once or twice a day.
Allow a minimum of one hour between wear time.
Do not exceed 20 minutes per application.